



¹³C-Urea Breath Test

For Detection of Helicobacter Pylori

“Gold Standard”



EASY

Simple to do, no invasive procedures, no messy collection cups, no painful needles



CONVENIENT

Can be done in your doctor's office or at a lab



FAST

The entire test process takes about 20 minutes

OPERATION PROCEDURE



1. Blow up the first collection bag



2. Take one ¹³C-urea tablet



3. Wait for 20 minutes



4. Blow up the second collection Bag



5. Test two bags of breath samples with an analyzer


APPLICATION

Applicable for all ages including pregnant women and children.



PRODUCT HIGHLIGHT

- > Sensitivity 95%
- > Specificity 100%
- > Painless and non-invasive
- > Analytical results generated within 3 minutes
- > Applicable for any age

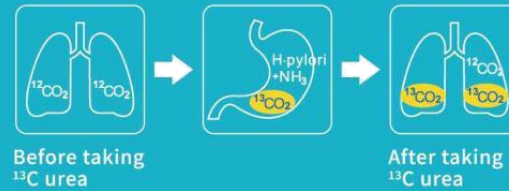

PRODUCT FEATURE

> High efficiency

Fast operation, 2-3 minutes for one set samples automatic batch testing.

> Simple

One-touch operation, automatic control system.

> Accurate

Good Repeatability. RSD<0.2%.

SPECIFICATIONS

1. Power Supply.	AC110-220V 50HZ
2. Power Consumption.	<50W
3. Range of CO Concentration.	0.5-6%
4. Sample Volume.	≥120ml
5. Dimension.	330mm x 450mm x 245mm
6. Weight.	15kg



MediTec srl

Sede Operativa:
Via Antonino di Sangiuliano, 99
Sant'Agata Li Battiati (CT)
www.meditecdiagnostici.com

